




The original waterproof protector

Helping patients shower and bathe with confidence

Waterproof Protection for Casts,
Air Walker Boots, Dressings
and PICC Lines/Midlines

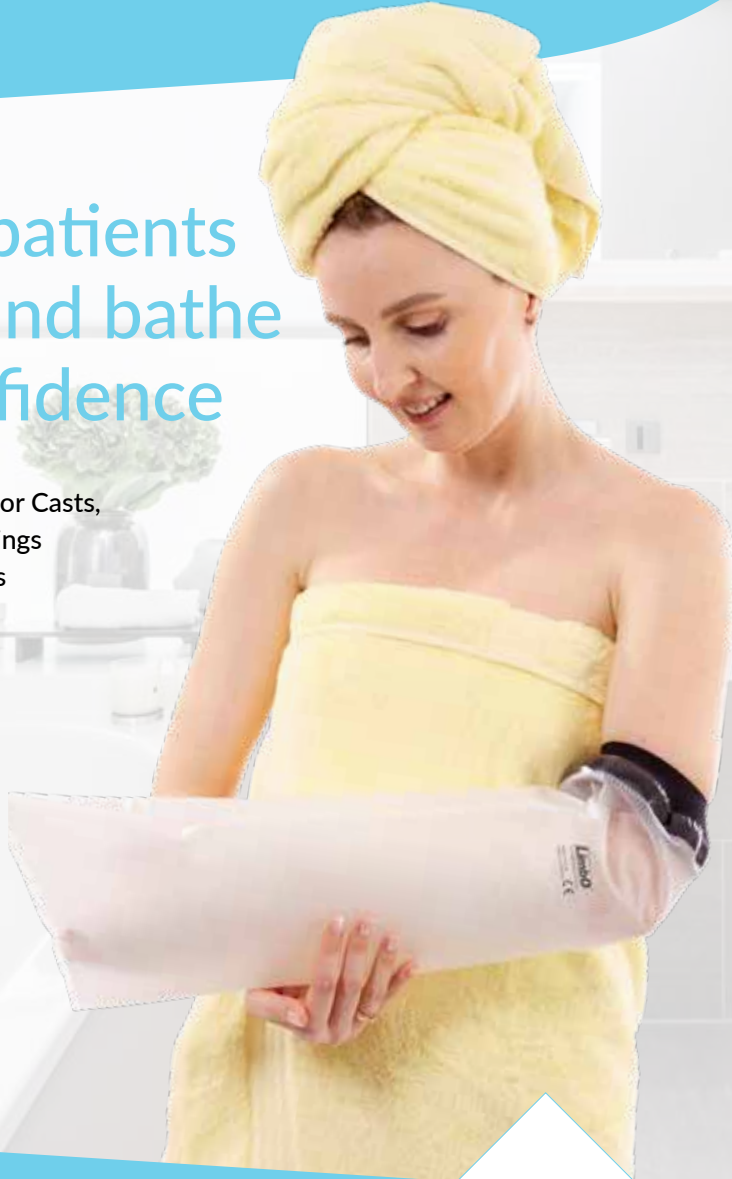

Proudly designed and
manufactured in the UK
since 1994.

 **Thesis**
Technology

Help restore patients' well-being with a LimbO waterproof protector

Thank you for your professional interest in the LimbO Waterproof Protector. This guide explains why the LimbO is recognised as a leading waterproof protection solution for patients and quickly assists you with product selection. We have also included a section about our outdoor cast protectors, cast accessories, mobility aids and a range of diabetic socks.

For further support and resources please visit www.limboproducts.co.uk or see our contact details on the back cover.



Contents

Benefits of the LimbO	3
Adult LimbOs	4 - 7
Elbow LimbOs (adult & child)	5
Child LimbOs	8 - 9
Cast / Dressing Accessories	10
Outdoor Cast / Dressing Protectors	11
Daily Living Aids and Diabetic Socks	12
Body Block	13
Size Guide	14 - 15
Contact Details	Back cover

"Patient use of the LimbO reduces the chance of wet dressings and casts and the associated problems of discomfort, clinical complications and delays to recovery".

Why is the LimbO the most recommended protector?

Helping the Patient

- More independence during recovery
- Easy to use
- Maintain personal hygiene
- Durable and submersible
- Reliable waterproof protection
- Adult and child sizes
- Easy to order via phone, web or Amazon
- Prices from £9.15
- Rapid delivery
- Latex free

Helping Your Team

- Reduce healthcare risks associated with wet dressings and wet casts
- Reduce avoidable NHS costs resulting from re-casting and re-dressing
- Easy size selection
- Certain models available on FP10 prescription
- Bespoke sizes available for unique cases
- Intended for single patient use
- 25 years as a trusted solution

Featuring the unique LimbO neoprene seal

- Flexible and comfortable
- Superior design providing a complete waterproof barrier against the skin
- Gentle non-constrictive seal, made with premium neoprene
- Guaranteed for twelve weeks with regular use
- Proudly designed and manufactured in the UK



Guaranteed waterproof protection for casts, dressings or PICC lines



Adult Half Arm

For casts or dressings below the elbow.

- Seals just above the elbow
- Suitable for the bath or shower



Height (cm)	Weight (kg)	Length (cm)	Circumference (cm)		Model
			Limb	Cast	
152+	Under 51	49	22-25	33	M50A
152+	51-70	53	25-29	40	M60
152+	70+	53	30-39	44	M67



Adult Full Arm

For casts or dressings covering the whole arm.

- Seals at the top of the upper arm
- Suitable for the bath or shower



Height (cm)	Weight (kg)	Length (cm)	Circumference (cm)		Model
			Limb	Cast	
152+	51-70	69	25-29	40	M60L
152+	70-108	69	30-39	44	M70
152+	108+	69	39-54	57	M87



Adult and Child Elbow

For PICC Lines, Midlines, or dressings on the elbow or upper arm.

- Seals above and below the elbow
- Ideal to support patients during IV treatment
- Suitable for the bath or shower
- Prescription options
- Child option



Height (cm)	Weight (kg)	Length (cm)	Circumference (cm)		Model	PIP Code
			Limb	Dressing		
Under 152	Under 51	12	22-25	33	M45S	N/A
152+	Under 51	26	22-25	33	M45	N/A
152+	51-70	26	25-29	40	M65	386-4303
152+	70-108	26	30-39	44	M75	386-4311
152+	108+	26	39-54	57	M85	N/A



Adult Hand

For light dressings on the hand.

- Seals on the forearm
- Not suitable for plaster casts or following thumb surgery. Select Adult Half Arm model instead



Height (cm)	Weight (kg)	Length (cm)	Circumference (cm)		Model
			Limb	Cast	
Slim to Medium Build		30	22-25	Bandage Only	MITT


The comfortable, trusted solution for staying dry in the shower or bath



Adult Half Leg

For casts or dressings below the knee.

- Seals just above the knee
- Suitable for use with diabetic ulcers
- Suitable for the bath or shower
- Prescription options

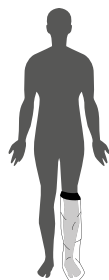


Height (cm)	Weight (kg)	Length (cm)	Circumference (cm) Limb	Cast	Model	PIP Code
165 to 183	64-102	73	39-54	57	M80	320-0045
Under 165	64-102	65	39-54	57	M80S	320-0052
183+	64-102	83	39-54	57	M80L	N/A
165 to 183	Under 64	73	35-39	44	M76	333-7581
Under 165	Under 64	65	35-39	44	M76S	333-7599
183+	Under 64	83	35-39	44	M76L	N/A
165 to 183	102-140	73	52-57	59	M180	320-0060
Under 165	102-140	65	52-57	59	M180S	320-0078
183+	102-140	83	52-57	59	M180L	N/A
152 to 183	140 +	73	65-90	94	M200HL	N/A



Adult Full Leg

For casts or dressings covering the whole leg.

- Seals at the top of the leg
- Suitable for use with diabetic ulcers
- Suitable for the bath or shower

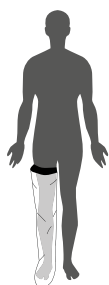

Height (cm)	Weight (kg)	Length (cm)	Circumference (cm) Limb	Cast	Model
165 to 183	64-102	96	52-65	70	M100
Under 165	64-102	85	52-65	70	M100S
183+	64-102	103	52-65	70	M100L
Under 183	Under 64	96	39-54	57	M80LL
152 to 183	102+	96	65-90	94	M200FL



Adult Knee

For light dressings on the knee.

- Seals above and below the knee
- For shower use only



Height (cm)	Weight (kg)	Length (cm)	Circumference (cm) Limb	Dressing	Model
165 to 183	64-102	38	39-54	57	M86

Adult Orthopaedic Boot

For orthopaedic boots. For the seal to be large enough to go over the boot, please select the appropriate size Adult Full Leg LimbO model.

- Seals at the top of the thigh
- Suitable for the bath or shower



Adult Foot

For light dressings below the ankle.

- Seals above the ankle bone
- Suitable for use with diabetic ulcers
- For shower use only
- FP10 prescription options available early 2021



Shoe Size	Circumference (cm) Limb	Dressing	Model
Max shoe size 11	20-25	33	M20
Max shoe Size 11	23-34	40	M25

Eliminate the need for replacing wet bandages or ruined plaster casts



Child Full Leg

For casts or dressings covering the leg.

- Seals at the top of the thigh
- Suitable for the bath or shower



Age	Weight (kg)	Height	Length (cm)	Circumference (cm) Limb	Cast	Model
3 yrs	Size is typical for age, select as you would for clothing		53	25-34	40	FL3
4-5 yrs			58	25-34	40	FL45
6-7 yrs			61	35-39	44	FL67
8-10 yrs			73	39-54	57	FL810
11-13 yrs			83	39-54	57	FL1113



Child Half Leg

For casts or dressings below the knee, on older children.

- Seals just above the knee
- Suitable for the bath or shower



Age	Weight (kg)	Height	Length (cm)	Circumference (cm) Limb	Cast	Model
8-10 yrs	Size is typical for age, select as you would for clothing		58	25-34	40	BK810
11-13 yrs			61	35-39	44	BK1113



Child Full Arm

For casts or dressings on the arm.

- Seals at the top of the arm
- Aqua blue junior style
- Suitable for the bath or shower



Age	Weight (kg)	Height	Length (cm)	Circumference (cm) Limb	Cast	Model
2-3 yrs	Size is typical for age, select as you would for clothing		31	16-22	29	FA23
4-5 yrs			43	16-22	29	FA45
6-7 yrs			49	22-25	33	FA67
8-10 yrs			58	22-25	33	FA810
11-13 yrs			58	25-34	40	FA1113



Child Half Arm

For casts or dressings below the elbow, on older children.

- Seals just above the elbow
- Slim and standard arm options
- Aqua blue junior style
- Suitable for the bath or shower



Age	Weight (kg)	Height	Length (cm)	Circumference (cm) Limb	Cast	Model
8-10 yrs-Slim	Size is typical for age, select as you would for clothing		36	16-22	29	BE810-40
8-10 yrs-Standard			36	22-25	33	BE810-50
11-13 yrs-Slim			43	16-22	29	BE1113-40
11-13 yrs-Standard			43	22-25	33	BE1113-50


Helpful accessories and outdoor protection for casts and dressings

Making life more comfortable



Arm Cast / Dressing Sleeve


Brighten up your arm cast with a soft lycra cover.

- Helps avoid snagging
- Adult and child sizes
- Available colours 



Adult Leg Cast / Dressing Sleeve


Keep your leg cast clean with a stylish lycra cover.

- Helps avoid snagging
- Available colours 



Toe Cozy


Keep toes clean and warm with a cosy fleece cover.

- Adjustable Velcro® straps
- Can be used with dressings too
- Available colours 



Oarsome Grips

Extra comfort for crutch handles.

- Helps avoid blisters
- Sold as a pair
- Available colours 

Keep clean and dry whilst outside



Adult Foot Cover

Keep your cast / dressing clean and dry whilst outside.

- Breathable waterproof fabric
- Adjustable Velcro® straps



Adult Mitt / Child Mitt

Keep your cast / dressing clean and dry whilst outside.

- Breathable waterproof fabric
- Adjustable Velcro® straps



Adult Wrist Cover

Keep your cast / dressing clean and dry whilst outside with exposed fingers for a better grip.

- Breathable waterproof fabric
- Adjustable Velcro® straps
- Left and right hand options



Sealskinz

Rugged outdoor sock for child's lower leg.

- Perfect for the garden or beach
- Waterproof and breathable



Diabetic Sock Collection - Socks that do not constrict blood flow on the lower leg

Prosox

- Stay up without applying pressure
- Welt-free
- Available in cotton or wool
- Anti-slip option available



Gentle Grip Socks

- Non-binding
- Available in black, grey or six vibrant colours as shown
- Extra wide oedema option



Sock Aid

- Pull on socks with minimum stretching
- Ribbed thumb grips
- Lightweight



Walk-Ease Walking Stick

- Adjustable length
- Handle designed for extra comfort
- Flexible ferrule to maximise contact with the ground
- Three styles



Stick and Stay (Suction Grab Rail)

- Easy to fit
- Ideal for travel
- Suitable for ceramic or glass
- Safety indicator



Helping you whilst you help patients

The bodyblock limb support provides a hygienic multidisciplinary, multipurpose support block for clinical environments.

bodyblock

With four sides creating different heights, the bodyblock is a versatile tool for delivering stable elevation during treatment. Simply flip it for a different height and reduce bending to treat patients. The bodyblock can be used to support the head, legs, heels or arms and can also be used as knee spacer. The bodyblock is lightweight, strong and easy to sterilise.



NO QUIBBLE GUARANTEE

We want users to be totally happy with their LimbO and the service they receive from us. The LimbO is guaranteed to last twelve weeks when used correctly. If the LimbO proves unsuitable for the intended application or fails to last twelve weeks, a refund will be given or replacement will be provided. This does not affect statutory rights.

Please note: The LimbO comes with full instructions and safety information. The LimbO is not intended for use on open or undressed wounds and should not be used in the sea or swimming pools.

Prices correct at June 2020.

Adult

(for quick size selection use height and weight. Use length and circumference points shown on diagram for extra guidance)

Limb Coverage	Full Product Description	Height (cm)	Weight (kg)	Limbo Length (cm)	Limb Circumference (cm)	Cast Circumference (cm)	Model	PIP Code	Diabetic Ulcers	PICC Lines	Page
Full	Adult Full Arm-Slim	152+	51-70	69	25-29	40 (maximum)	M60L				4
Full	Adult Full Arm-Medium	152+	70-108	69	30-39	44 (maximum)	M70				4
Full	Adult Full Arm-Large	152+	108+	69	39-54	57 (maximum)	M87				4
Half	Adult Half Arm-Slimmer Adult	152+	Under 51	49	22-25	33 (maximum)	M50A				4
Half	Adult Half Arm-Slim to Medium	152+	51-70	53	25-29	40 (maximum)	M60				4
Half	Adult Half Arm-Medium to Large	152+	70+	53	30-39	44 (maximum)	M67				4
Elbow	Adult Elbow-Extra Slim	152+	Under 51	26	22-25	33 (maximum)	M45			●	5
Elbow	Adult Elbow-Slim to Medium	152+	51-70	26	25-29	40 (maximum)	M65	386-4303		●	5
Elbow	Adult Elbow-Medium to Large	152+	70-108	26	30-39	44 (maximum)	M75	386-4311		●	5
Elbow	Adult Elbow-Extra Large	152+	108+	26	39-54	57 (maximum)	M85			●	5
Full	Adult Hand	Slim to Medium Build		30	22-25	Bandage Only	MITT				5
Full	Adult Full Leg	165 to 183	64-102	96	52-65	70 (maximum)	M100				6
Full	Adult Full Leg-Short	Under 165	64-102	85	52-65	70 (maximum)	M100S				6
Full	Adult Full Leg-Long	183+	64-102	103	52-65	70 (maximum)	M100L				6
Full	Adult Full Leg-Slim	Under 183	Under 64	96	39-54	57 (maximum)	M80LL				6
Full	Adult Full Leg-Extra Heavy Leg	152 to 183	102+	96	65-90	94 (maximum)	M200FL				6
Half	Adult Orthopaedic Boot	For any Orthopaedic Boot please select a full leg LimbO									6
Half	Adult Half Leg	165 to 183	64-102	73	39-54	57 (maximum)	M80	320-0045	●		6
Half	Adult Half Leg-Short	Under 165	64-102	65	39-54	57 (maximum)	M80S	320-0052	●		6
Half	Adult Half Leg-Long	183+	64-102	83	39-54	57 (maximum)	M80L		●		6
Half	Adult Half Leg-Slim	165 to 183	Under 64	73	35-39	44 (maximum)	M76	333-7581	●		6
Half	Adult Half Leg-Slim-Short	Under 165	Under 64	65	35-39	44 (maximum)	M76S	333-7599	●		6
Half	Adult Half Leg-Slim-Long	183+	Under 64	83	35-39	44 (maximum)	M76L		●		6
Half	Adult Half Leg-Large	165 to 183	102-140	73	52-57	59 (maximum)	M180	320-0060	●		6
Half	Adult Half Leg-Large-Short	Under 165	102-140	65	52-57	59 (maximum)	M180S	320-0078	●		6
Half	Adult Half Leg-Large-Long	183+	102-140	83	52-57	59 (maximum)	M180L		●		6
Half	Adult Half Leg-Extra Heavy Leg	152 to 183	140 +	73	65-90	94 (maximum)	M200HL		●		6
Knee	Adult Knee	165 to 183	64-102	38	39-54	57 (maximum)	M86				7
Full	Adult Foot-Slim to Medium	Max shoe size 11			20-25	33 (maximum)	M20	TBC 2021	●		7
Full	Adult Foot-Medium to Large	Max shoe Size 11			23-34	40 (maximum)	M25	TBC 2021	●		7

Conversion Tables	
Stone	Kilos
8	51
8.5	54
9	57
9.5	60
10	64
10.5	67
11	70
11.5	73
12	76
12.5	79
13	83
13.5	86
14	89
14.5	92
15	95
15.5	98
16	102
16.5	105
17	108
17.5	111
18	114
18.5	118
19	121
19.5	124
20	127


Feet	cm
4	122
4.6	137
5	152
5.6	168
6	183
6.6	198
7	214

Child

(for quick size selection use clothing size. Use length and circumference points shown on diagram for extra guidance)

Full	Child Full Arm-2-3 yrs			31	16-22	29 (maximum)	FA23				9
Full	Child Full Arm-4-5 yrs			43	16-22	29 (maximum)	FA45				9
Full	Child Full Arm-6-7 yrs			49	22-25	33 (maximum)	FA67				9
Full	Child Full Arm-8-10 yrs			58	22-25	33 (maximum)	FA810				9
Full	Child Full Arm-11-13 yrs			58	25-34	40 (maximum)	FA1113				9
Half	Child Half Arm-8-10 yrs-Slim			36	16-22	29 (maximum)	BE810-40				9
Half	Child Half Arm-8-10 yrs-Standard			36	22-25	33 (maximum)	BE810-50				9
Half	Child Half Arm-11-13 yrs-Slim			43	16-22	29 (maximum)	BE1113-40				9
Half	Child Half Arm-11-13 yrs-Standard			43	22-25	33 (maximum)	BE1113-50				9
Elbow	Child Elbow	Under 152	Under 51	12	22-25	33 (maximum)	M45S			●	5
Full	Child Full Leg-3 yrs			53	25-34	40 (maximum)	FL3				8
Full	Child Full Leg-4-5 yrs			58	25-34	40 (maximum)	FL45				8
Full	Child Full Leg-6-7 yrs			61	35-39	44 (maximum)	FL67				8
Full	Child Full Leg-8-10 yrs			73	39-54	57 (maximum)	FL810				8
Full	Child Full Leg-11-13 yrs			83	39-54	57 (maximum)	FL1113				8
Half	Child Half Leg-8-10 yrs			58	25-34	40 (maximum)	BK810				8
Half	Child Half Leg-11-13 yrs			61	35-39	44 (maximum)	BK1113				8

Measuring Guide

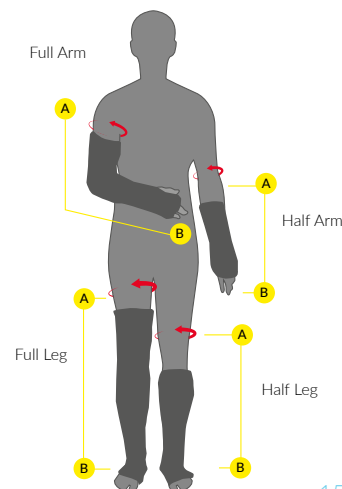

Here to help you, your team and your patients

Our clinical support team is on hand to help you with:

- Questions about the LimbO
- Supplying posters, leaflets and literature for your clinic/hospital
- Prescription request forms
- Samples or demonstrations of the LimbO for your team
- Wholesale/volume purchasing
- Support to assist with CPD
- Details of international re-sellers and distributors

For Clinical Support:

- Call: 01243 578755
- Email: info@limboproducts.co.uk
- Visit: www.limboproducts.co.uk

If referring patients directly, please ask them to contact our customer services team on: 01243 573417

Rapid delivery: Same day despatch for orders up to 4.00pm weekdays and 11.00am on Saturdays. Standard P&P from £2.95.



LimbOProducts



@LimbO_Products

Thesis Technology Products Ltd,
Brooks Green Farm, Brooks Lane,
Bosham, Chichester, PO18 8JX, UK

